Clario *Toni*

SUCTION PUMP

- EN Instructions for use
- DE Gebrauchsanweisung
- FR Mode d'emploi
- IT Istruzioni per l'uso
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- FI Käyttöohjeet
- NO Bruksanvisning
- ES Instrucciones de uso
- PT Instruções de utilização

Table of contents

Congratulations	4
1 Warnings and safety instructions	5
Safety instructions	7
2 Power supply and battery operation	8
Battery low	8
Battery empty	8
3 Description	9
Introduction	9
Intended use/indications	9
Contraindications	9
Intended user	9
Intended patient population	9
Important note	9
4 Overview	10
Definition of vacuum	10
Versions of the pump	10
Main elements of the pump	10
Position of the membrane	11
Valve discs in the SafetyChamber	11
Operating elements and battery status	11
5 Installation	12
Check initial delivery	12
6 Preparation for use	13
Checks before use	13
Assembly of the collection system	13
Assembly of the SafetyChamber	14
Attaching the collection system to the pump	14
7 Operating instructions	15
Connect Clario Toni to mains power	15
Functional check with water	16
Alternative: functional check with the optional vacuum gauge	17
Changing vacuum level	18
Placing out of operation after use	18
8 Emptying the canister	19
9 Replacing the valve discs	20

10	Replacing the rechargeable battery	21
11	Battery data and test	22
	Duration of pumping	22
	Charging time (charged up to 100%)	22
	Battery status	22
	Testing the rechargeable battery	22
12	Battery calibration	23
	Calibrating NiMH rechargeable battery	23
	Storage of the rechargeable battery	23
13	Troubleshooting	24
	Motor not running	24
	Pump is too loud	24
	Insufficient vacuum	24
	Running time from battery insufficient	24
	No LED lit	24
	LED lit up	24
	LED blinks slowly	24
	LED blinks rapidly	24
14	Warranty and maintenance	25
	Warranty	25
	Maintenance/routine check	25
15	Disposal	25
16	Cleaning guidelines	26
	General notes	26
	Medela recommended surface cleaning agents for pump housing	26
	Water	26
	Cleaning/disinfection machines	26
	Disposable products	26
17	Accessories overview	28
18	Accessories legend	29
19	Signs and symbols	30
20	Technical specifications	32
21	Technical documentation	33
	Technical documentation/EMC	326

Congratulations

By choosing the Clario Toni, you have acquired a high-quality suction pump which is adaptable to your needs on order of a health professional/ physician only. This product provides you with a unique threefold hygienic design and optimal protection from an overflow. It is operated by a membrane system with an electric motor. Using the 3 vacuum settings, the suction strength can be adjusted to the required level. As all Medela suction pumps, this pump provides reliable and simple suction. Its simple handling and cleaning as well as the safety features are additional advantages you receive. Contact us – we will be pleased to advise you.

1 Warnings and safety instructions

WARNINGS

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTIONS

Indicates a potentially hazardous situation which, if not avoided, could result in minor or moderate injury.

Safety related tip

Indicating useful information about the safe use of the device.

The Clario Toni is approved exclusively for the use as described in these instructions for use. Medela can only warranty the safe functioning of the system when the Clario Toni is used in combination with the original Medela accessories. See chapters 17/18.

The Clario Toni is EMC-tested in conformity with the requirements of IEC 60601-1-2:2007 and can be used in the vicinity of other EMC-tested devices that fulfil the requirements as outlined in the IEC 60601-1-2 standard. Untested HF sources, radio networks or the like can impair the function of the device and should not be operated in combination with the Clario Toni.

Please read and observe these warning and safety instructions before operation. These instructions for use must be kept with the device for later reference.

Please note that these instructions for use are a general guide for the use of the product. Medical matters must be addressed by a physician.

Medela considers herself only responsible for the effect on BASIC SAFETY, reliability and performance of the Clario Toni if it is used in accordance with the instructions for use.

CAUTION: Federal US law restricts this device to sale by or on the order of a health professional.

Subject to change.

WARNINGS

- For use only by medically trained persons who have been adequately trained in suction procedures and in the use of aspirators.
- For patients for whom a breakdown of the device can lead to a critical situation (e.g. patients with acute dyspnoea or severe catarrhal congestion), a replacement device must always be available.
- The device must not be used for suctioning explosive, easily flammable or corrosive liquids.
- The connecting tubing supplied with the device must never come into direct contact with the suction area. A sterile suction catheter must always be used (risk of infection).
- Before cleaning the device, pull the plug out of the fixed mains socket.
- No modification of this equipment is allowed.
- Do not connect this device to a passive drainage tube.
- Consult the indications for use and consider risk factors and contraindications before using the Clario Toni. Failure to read and follow all instructions in this manual prior to use may result in serious or fatal injury of the patient.

CAUTIONS

- Incorrect use can cause pain and injury to the patient.
- Do not use sterile accessories when the sterile packaging is damaged.
- The use of mobile telephones, LAN/WLAN, walkie-talkies (two-way radios) and cordless telephones sets can affect the Clario Toni pump. A safety distance of min. 3.3ft (1 m) to the Clario Toni pump is recommended.
- Portable and mobile RF communications equipment can affect medical devices.

Safety related tip

- For safety tests, it is assumed that the device is serviced and repaired throughout its service life in accordance with the service manual. Because the Clario Toni is a device of safety class II (EN IEC 60601-1, A1, A2:1995), the safety tests are confined to visual inspection of the housing and power cord. This test must be carried out before each use.
- Testing the patient leakage current: Please see service manual for details.
- The protection of the Clario Toni against the effects of the discharge of a cardiac defibrillator is dependent upon the use of appropriate cables.
- Separation from the mains is only assured through the disconnection of the mains plug and the fixed mains socket.
- Before you plug in the device, please check that your local power supply is the same as the voltage given on the specification plate.

Safety instructions

- The Medela Clario Toni is a medical device that requires special safety measures in regard to EMC. It must be installed and put into operation in accordance with the attached EMC information.
- In each of the following cases, the device must not be used and it must be repaired by Customer Services:
 - if the power cord or the plug are damaged
 - if the device is not functioning perfectly
 - if the device is damaged
 - if the device shows clear safety defects.
- Keep the power supply cord away from hot surfaces.
- The mains plug and the on/off switch must not come into contact with moisture.
- Never pull the mains plug out of the fixed mains socket by pulling on the power supply cord!
- Never leave the device unattended when it is switched on.
- The pump must stand upright during use.
- Never place the power supply cord around your neck.
- Never use the device at high room temperatures, while bathing or showering, if you are very tired or in an environment where there is a risk of explosion.
- Never place the device in water or other liquids.
- When using single use, sterile products, please note that they are not intended to be reprocessed. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross contamination.
- Contact your local Medela customer service representative for assistance with product operations.
- Wear gloves for all operations.

These instructions for use must be kept for later reference.

2 Power supply and battery operation

The Clario Toni is a mains- and battery powered suction pump. Before you plug in the device, please check that your local power supply is the same as the voltage given on the specification plate. Used as described in this user guide, the Clario Toni AC/DC model has a maximum of 50 minutes of battery time before it is necessary to recharge. If the Clario Toni AC/DC model is not in use, the battery must be calibrated every 60–90 days.

Battery low

During battery operation a slow blinking of the LED indicator indicates, that the rechargeable battery is soon discharged. The function of the Clario Toni is maintained for a short time (~10 minutes), but the battery must be recharged as soon as possible.

Battery empty

As soon as the rechargeable battery is empty, the pump comes to a stop. The proper function of the pump is no longer maintained. The rechargeable battery must be recharged or replaced.

3 Description

Introduction

The Clario Toni is a high-quality suction pump. It is powered by the well-proven membrane system. It ideally combines easy handling and cleaning with safety features to ensure optimal operation. Use only according to the instructions from your doctor and after getting instructions from trained medical specialists.

Intended use

The Clario Toni suction pumps are indicated for aspiration and removal of secretions, bodily fluids and foreign objects from a patient's airway or respiratory support system in the nasal, pharyngeal and tracheal areas of pediatric and adult patients. The suction pump is suitable for use at home, in the hospital, in the car or outdoors.

Indications

The Clario Toni suction pump is indicated to clear the airways either through the nose, mouth or tracheostomy tube.

Contraindications

The device is not suitable for use for prolonged suctioning.

Intended user

The Clario Toni should only be operated by properly instructed users. These persons must not be hard of hearing or deaf and must have adequate visual faculty. The instruction has to be refreshed at least once a year.

Intended patient population

The Clario Toni is intended to be used on patients only exhibiting conditions as described in the indications for use.

Important note

Compliance with proper surgical procedures and techniques is the responsibility of the physician. Each physician must evaluate the appropriateness of the treatment based on his own knowledge and experience.

Service life

The service life of the device is three years.

4 Overview

Definition of vacuum

By the application of medical aspiration devices, vacuum is normally given as the difference (in absolute figures) between absolute pressure and atmospheric pressure or as negative values in Kilopascal (kPa). In this document, the indication of -10 kPa for example always refers to a pressure range in kPa below atmospheric ambient pressure (according to EN ISO 10079:1999).

Versions of the pump

Clario Toni AC/DC

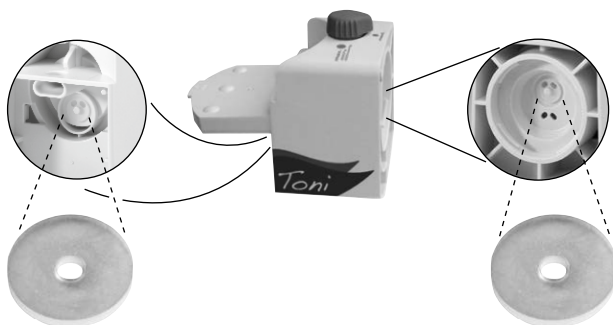
Main elements of the pump

Position of the membrane

Valve discs in the SafetyChamber

Operating elements and battery status

Vacuum regulator

On/off button

Rechargeable battery LED

- | | |
|-----------------------|--|
| not lit | Battery is fully charged |
| lit up | – Battery is being charged
– Pump is plugged in |
| blinks slowly | Battery is soon empty |
| blinks rapidly | Battery is defective |

5 Installation

Check initial delivery

Check the delivery package of the Clario Toni for completeness and general condition.

Clario Toni versions

REF	Clario Toni AC/DC	014.00XX
	Clario Toni Deluxe	014.00XX

Disposable fingertip tubing

REF	074.0003
------------	----------

Power adapter

REF	014.0004
------------	----------

Power cord

099.0011 (Euro)
014.0007 (UK)
014.0008 (AUS/NZ)
014.0009 (USA)

Instructions for use

REF	200.8400
------------	----------

6 Preparation for use

WARNINGS

For use only by medically trained persons who have been adequately trained in suction procedures and in the use of aspirators. Wear gloves for all operations.

CAUTIONS

- The Clario Toni must remain in an upright position during use.
- Sterile accessories must be checked on the integrity of the packaging before use.
- Non sterile and reusable accessories must be cleaned, disinfected and/or sterilized according to the Medela cleaning guide (product code 200.2391).

6.1 Checks before use

- Check the Clario Toni system before use for damage of the power cord or plug, obvious device damage or safety defects and proper functioning of the device.
- Check for completeness and general condition of the Clario Toni delivery package.
- Make sure that the rechargeable battery is calibrated according to chapter 12.
- Check all accessories prior to use:
 - Canister and lid for cracks, brittle and flawed spots. Replace if necessary.
 - Tubing for cracks, brittle areas and that connectors are firmly attached. Replace if necessary.

6.2 Assembly of the canister

1. 1.1 Attach the floater to the canister.

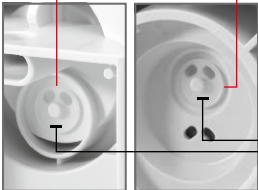
2. 2.1 Attach the lid to the canister. Mind the markings on the front of the canister and the lid.

6.3 Assembly of the SafetyChamber

1.

1.1 Open the lid of the SafetyChamber.
2.

2.1 Make sure that both valves (silicone discs) are attached to the SafetyChamber. See chapter 4.
3.

3.1 Make sure that the lid of the SafetyChamber is closed well.
4.

4.1 Hold the SafetyChamber and tilt 45° to the left to attach it to the motor unit.

4.2 Turn the SafetyChamber to an upright position until you hear a click.

6.4 Attaching the canister to the pump

1.

1.1 Attach the canister to the pump. Press firmly.
2.

2.1 Attach the disposable fingertip tubing to the suction port.

7 Operating instructions

CAUTIONS

The Clario Toni is to be set up in such a way, that a separation from the mains supply can be easily managed. Calibrate the rechargeable battery before the first use.

WARNINGS

For use only by medically trained persons who have been adequately trained in suction procedures and in the use of aspirators. Wear gloves for all operations.

7.1 Connect Clario Toni to mains power

1. Check the pump before use following the instruction in chapter 6.1.

2. **If the pump is connected to a fixed mains socket:**

Plug in the mains plug of the power cord to a fixed mains socket.

or

If the pump is operated with a 12 VDC power source:

Connect the optional 12 VDC connection cable to the pump and to the 12 VDC power source.

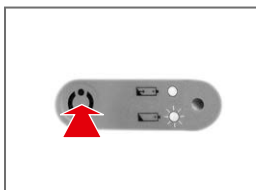
or

If the pump is operated with the rechargeable battery:

No connection to a power source is needed. Make sure that the battery was charged for five hours.


7.2 Functional check with water

1.


1.1 Press the On/off button to switch on the pump.

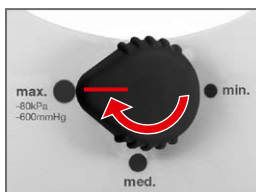2.


2.1 Set vacuum regulator to position «max.».

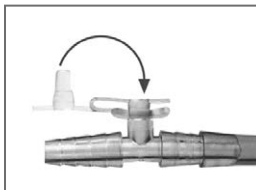3.

3.1 Attach the fingertip tubing.
See chapter 6.4, step 2.1.
3.2 Plug fingertip port.

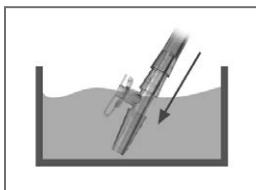4.

4.1 Place fingertip into water
and check if there is
suction.

5. If there is suction, proceed with step 6. If not, see chapter 13.

6. Use catheter and vacuum setting as prescribed by a physician.


7.3 Alternative: functional check with the optional vacuum gauge

1.


1.1 Attach the optional vacuum gauge to the suction port.

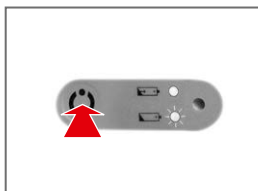2.

2.1 Press the On/off button to switch on the pump.

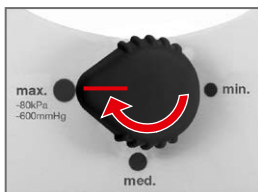3.

3.1 Set vacuum regulator to position «max.».

Altitude above sea level:	Max. Vacuum:
+ 2000 m	– 63 kPa – 471 mmHg
+ 1000 m	– 71 kPa – 532 mmHg
+ 500 m	– 76 kPa – 566 mmHg
0 m	– 80 kPa – 600 mmHg

(Tolerance: – 15 %)

4.

4.1 Compare the vacuum with the values in the table on the left.

5. If the vacuum is ok, proceed with step 6. If not, see chapter 13.

6. Attach the fingertip tubing.

7. Use catheter and vacuum setting as prescribed by a physician.

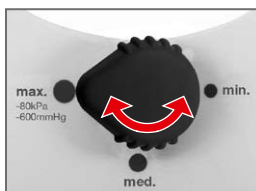
WARNINGS

For use only by medically trained persons who have been adequately trained in suction procedures and in the use of aspirators. Wear gloves for all operations.

7.4 Changing vacuum level


1.

1.1 Set vacuum regulator to the necessary position.

7.5 Placing out of operation after use

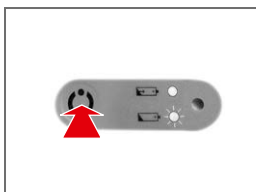1.

1.1 Press the On/off button to switch off the pump.

2. **If the pump is connected to a fixed mains socket:**

Disconnect the mains plug from the fixed mains socket.

or

If the pump is connected to a 12 VDC power source:

Disconnect the 12 VDC plug from the 12 VDC power source.

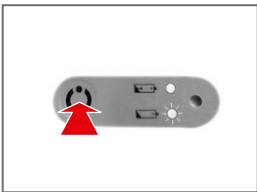
3. Clean and disinfect the Clario Toni. See chapter 16.

8 Emptying the canister

Safety related tip

On a daily basis or latest when the fluid level reaches 550ml and the floater stops suction into the canister the canister should be emptied.

1. 
 - 1.1 Press the On/off button to switch off the pump.
2. 
 - 2.1 Disconnect the disposable fingertip tubing from the suction port.
 - 2.2 Remove canister from the pump.
3. 
 - 3.1 For transportation always seal the canister with the plugs.
4. Empty canister and dispose of fingertip tubing in accordance with local guidelines and inhouse guidelines.
5. 
 - 5.1 Attach a clean canister.
 - 5.2 Attach a new disposable fingertip tubing to the suction port.
6. Switch on the pump and set the vacuum as prescribed by a physician.

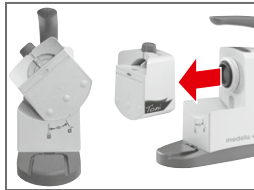
9 Replacing the valve discs

1.

1.1 Remove the canister.

2.

2.1 Turn SafetyChamber 45° to the left.

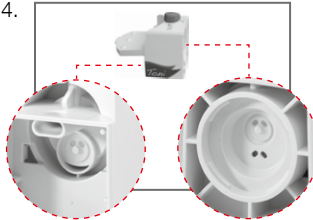
2.2 Remove SafetyChamber by pulling it off the pump.

3.

3.1 Open the SafetyChamber.

4.

4.1 Replace defective valve discs. Make sure that the new valve discs are attached correctly.

5. Attach the SafetyChamber to the pump as described in chapter 6.3.

10 Replacing the rechargeable battery

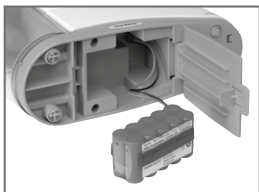
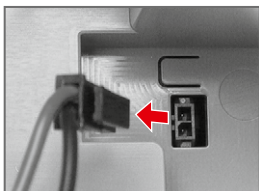
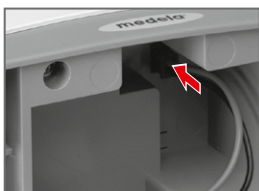
WARNINGS

Before replacing the rechargeable battery, switch off the Clario Toni and pull the mains plug from Clario Toni out of the fixed mains socket.

Before this operation, pull the mains plug out of the fixed mains socket.

1.  1.1 Open the lid of the battery compartment on the bottom of the Clario Toni.
2.  2.1 Detach the battery cable by pulling the plug.
3. Remove the old rechargeable battery.
4. Insert the new rechargeable battery.
5.  5.1 Reattach the battery cable by plugging in the plug.
6. Close the lid of the battery compartment. Make sure that the lid is closed well and that the battery cable is not clamped or pinched.
7. Calibrate and test the rechargeable battery (see chapters 11 / 12).

11 Battery data and test

CAUTIONS

Only charge the rechargeable battery when the charging indicator blinks slowly.
The rechargeable battery is charged during mains operation.

Duration of pumping

If the battery is fully charged, the operating time at maximum vacuum is approximately
> 50minutes.

Charging time (charged up to 100 %)

Approx. 5 hours.


Battery status

not lit	Battery is fully charged
lit up	– Battery is being charged – Pump is plugged in
blinks slowly	Battery is soon empty
blinks rapidly	Battery is defective

Testing the rechargeable battery


1. Make sure that the rechargeable battery is fully charged.
2. Pull out the mains plug from the fixed mains socket.

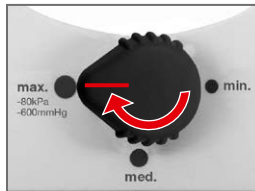3.

3.1 Set vacuum regulator to position «max.».

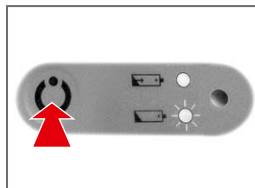4.

4.1 Switch on the Clario Toni.
The pump will start running.

5. Measure the time until the pump comes to a stop.
The battery is ok, when the the running time is
> 50minutes.

**If the running time is
< 50minutes, repeat test
or calibrate the battery
(see next page).**

12 Battery calibration

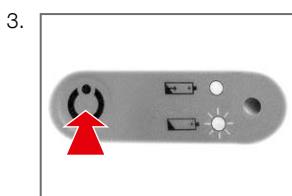
CAUTIONS

Calibrate the rechargeable battery before first use and after storage duration > 2 months.
Replace battery if the calibration is not successful.

Calibrating NiMH rechargeable battery

1. Make sure that the rechargeable battery is fully charged.
2. Pull out the mains plug from the fixed mains socket.

3.1 Switch on the Clario Toni.
The pump will start running.

4. Let the pump run until the rechargeable battery is empty and the pump comes to a stop.
5. Connect the Clario Toni to a fixed mains socket.
The rechargeable battery is being charged.
6. Wait until the charging indicator goes out.
7. Repeat step 1 to 6. The Clario Toni AC/DC is then ready to use.

Storage of the rechargeable battery

In order to keep the self-discharge at a minimum, store the Clario Toni and the replacement rechargeable batteries at temperatures below 25 °C (77 °F). Repeat the calibration process every 60–90 days.

13 Troubleshooting

Motor not running

Check if:

- the Clario Toni is switched on.
- the mains voltage is correct and if the mains plug is inserted correctly into the fixed mains socket.
- the internal battery is charged.

Pump is too loud

Check the position and seal of the valve discs inside the SafetyChamber.

Insufficient vacuum

Check if:

- the vacuum regulator is set correctly.
- the tubings are not defective or broken. If necessary, replace.
- all plug-in connections are tight.
- the suction canister/lid have no cracks, brittle areas, discolouration. If necessary, replace.
- the seal of the valve discs inside the SafetyChamber is tight.
- the membrane is clean and has no tears.

Running time from battery insufficient

Calibrate the rechargeable battery. See chapter 12.

No LED lit

The rechargeable battery is fully charged.

LED lit up

- The rechargeable battery is being charged.
- Pump is plugged in.

LED blinks slowly

The rechargeable battery is soon empty. The battery power is sufficient for about 10 minutes of pumping.

LED blinks rapidly

The rechargeable battery is defective. For replacing the defective battery see chapter 10.

14 Warranty and maintenance

Warranty

Medela AG warrants the device will be free from defects in materials and workmanship for a period of 2 years (6 months for the rechargeable battery) from the date of delivery.

Faulty material will be replaced free of charge during this period if not resulting from abuse or misapplication. This will not apply to parts subject to wear and tear in use. To ensure compliance with this warranty as well as optimum service from Medela products, we recommend the exclusive use of Medela accessories with our appliances.

In no event shall Medela AG be liable for claims which exceed the scope of warranty described including liability for consequential damages, etc. The right to the replacement of faulty parts will not be recognized by Medela if any work has been carried out on the pump by unauthorized persons. This warranty is subject to the appliance being returned to a Medela service centre.

Maintenance/routine check

Routine checks and service work are only to be carried out by positions authorised by Medela. The routine check is to be carried out 1 x per year (see service manual). The service manual, wiring diagrams, and descriptions are available upon request from Medela.

15 Disposal

The Clario Toni comprises metals and plastics and should be disposed of in accordance with the European directives 2002/95/EC and 2002/96/EC. Additional, local guidelines must also be observed. With the AC/DC versions, the electronic components and the rechargeable battery must be disposed of separately, in accordance with the local regulations.

Please take care that you dispose the Clario Toni and its accessories in accordance with your local disposal guidelines.

User information for the disposal of electrical and electronic equipment

This symbol means that the electrical and electronic equipment must not be disposed as normal household refuse. A correct disposal of this device protects and prevents possible damage to the environment or human health. For more information about the disposal contact the manufacturer, your local caregiver or healthcare provider. This symbol is only valid in the European Union. Please respect the relevant state laws and rules in your country for the disposal of electrical and electronic equipment.

16 Cleaning guidelines

WARNINGS

After each use, the parts that had been in contact with the aspirated secretions are to be cleaned, disinfected, sterilised or disposed of **according to the table on the next page**.

Before cleaning the device, pull the mains plug out of the fixed mains socket.

General notes

- These are general recommendations only that may be adjusted individually, based on the hospital's specific directives and cleaning practices and policies.
- Detailed information according to Medela cleaning instructions (product code 200.2391).
- Wear protective gloves for cleaning/disinfection.
- Dispose of fluids such as blood and secretions and the parts contaminated with them in accordance with internal hospital guidelines.

Medela recommended surface cleaning agents for pump housing

- **Hexaquart plus** (aldehyde-free), B. Braun Medical AG, www.bbraun.ch
- **Perform** [concentrate], Schülke & Mayr, www.schuelkemayr.com

Water

Use only the purest quality of water for cleaning. Water hardness is a serious consideration since deposits left on medical products may not be properly decontaminated. Use deionised water in order to reduce this problem. The final rinse water is bacterial free and contains no endotoxins.

Cleaning/disinfection machines

Can be used to disinfect parts from the table on the next page. A hot water rinse (maximum temperature 100 °C) may provide a medium-to high level of disinfection. Every section of the constituent parts must be accessible in order to ensure efficient cleaning. We recommend using a cleaning/disinfecting machine that has been approved by the Robert Koch- Institute and complies with ISO 15883. Recommended temperature for noncritical medical devices (i.e. those that only come into contact with uninjured skin) is 90 °C for 1 minute. The time is increased to 5 minutes for all medical devices that are considered to be critical.

Disposable products

These are single use products not intended to be reprocessed. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross contamination.


Canister, SafetyChamber	Pump housing Mains cable	
x	x	1. Disassembly Separate all individual parts before cleaning, disinfecting and sterilising.
x		2. Cleaning canister, SafetyChamber Clean components in hot water (60–70 °C) containing a detergent with a pH range between 6.0 and 8.0 only, in order to avoid damaging the instruments and containment devices. Enzymatic detergents help with the removal of organic matter, such as blood. Detergents should be used according to their manufacturer's recommended concentration levels. Some alkaline detergents have been formulated to be safe for reprocessing medical devices. The manufacturers of these detergents should provide information about specific materials that may be damaged by their detergents. Soak all parts thoroughly with warm, soapy water or in enzymatic detergent for 1–5 minutes. 1. Remove visible dirt with a cleaning tool – general purpose cleaning brushes, such as pipe cleaners or non-abrasive lint cloths. Brushes and pipe cleaners should fit snugly but still be able to be moved around easily within the area to be cleaned. Rinse thoroughly in clear water. Allow to dry 2. Check the instruments for visible dirt and repeat these steps if necessary.
	x	3. Cleaning pump housing, mains cable Wipe with detergent mentioned above. Use clean cloth to dry.
x		4. Disinfect Soak at room temperature for 30 minutes in a disinfection/cleaning solution (A). After the reaction time, use water to rinse the solution residue from the individual parts and rinse the tubing. Rinse for at least 2 minutes with the cleaning solution (B). (A) fluid aldehyde-free disinfection solution with cleaning effect for use as a soaking bath, bactericide, fungicide, limited virucide, with good material compatibility for stainless steel, non-ferrous material and plastics including silicone, slightly alkaline. e.g. 2 % neodisher® Septo MEDsolution (B) Fluid, pH-neutral, enzymatic cleaning solution for the treatment of instruments by machine or manually with very good material compatibility for stainless steel, non-ferrous material and plastics including silicone. or use cleaning/disinfecting machine. or use a dishwasher.
x		5. Storing components After disinfection store the components on a clean and dry cloth.

17 Accessories overview

WARNINGS

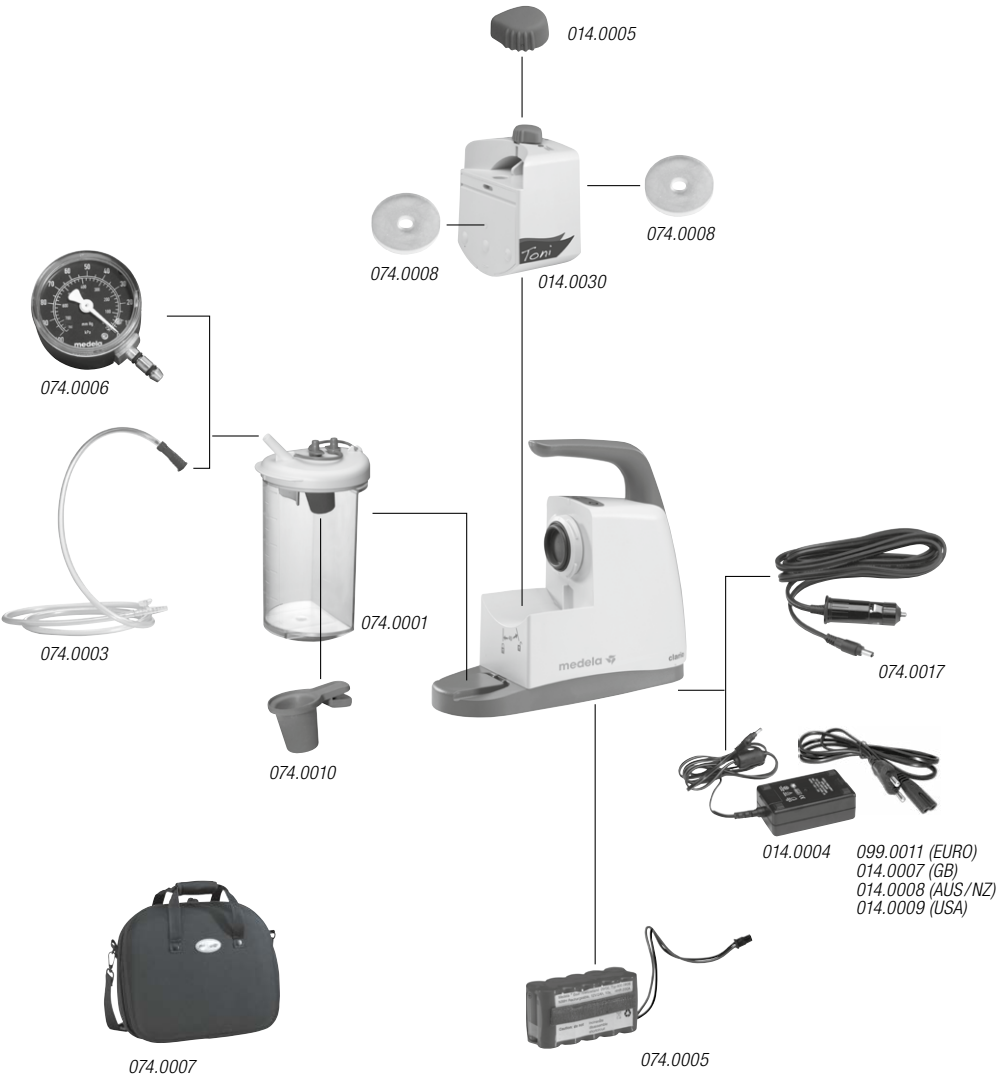Clario Toni was verified in combination with the accessories listed below. For a correct and safe operation use Clario Toni with these accessories only. Further information is supplied on the instruction sheet of the individual accessory.

18 Accessories legend

Safety related tip

If the pump is used together with non-Medela applied parts, they must:

- have CE mark and (if necessary) local registration
- be able to be attached to Medela accessories safely without impacting the performance of the pump.

Note: when combining Medela parts and non-Medela parts, you take over the responsibility of the entire system.

REF

014.0030	SafetyChamber
014.0005	Vacuum regulator
074.0008	Valve discs (10pcs.)
074.0001	Canister set
074.0010	Floater (10pcs.)
074.0003	Disposable fingertip tubing
074.0005	Rechargeable battery
014.0004	Power adapter
099.0011	Power cord (EURO)
014.0007	Power cord (GB)
074.0007	Carrying bag
074.0017	Car connection cable for 12V DC
074.0006	Vacuum gauge

19 Signs and symbols

0123 This symbol indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

This symbol indicates the class of the pump.

This symbol indicates a class II device.

IP21

This symbol indicates the protection against harmful effects due to ingress of solid foreign objects and against harmful effects due to the ingress of water.

This symbol indicates that the device should not be used after the end of the year and month shown.

This symbol indicates a single use device. Do not reuse the device.

REF

This symbol indicates manufacturer's catalogue number.

SN

This symbol indicates manufacturer's serial number.

LOT

This symbol indicates manufacturer's batch code.

STERILE EO

This symbol indicates the device is sterilized using ethylene oxide.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the compliance with additional USA and Canada safety requirements for medical electrical equipment.

This symbol indicates the legal specifications of the pump.

This symbol indicates to follow instructions for use.

This symbol indicates to consult instructions for use.

This symbol indicates a CAUTION or WARNING associated with the device.

This symbol indicates safety related tip.

This Symbol indicates a type BF applied part.

This symbol indicates MR UNSAFE.

This symbol indicates that interferences may occur in the vicinity of equipment marked with this symbol.

This symbol indicates the manufacturer.

This symbol indicates the date of manufacture (four digits for the year and two digits for the month).

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to keep the device dry.

pcs This symbol indicates number of items.

This symbol indicates do not use the device if package is damaged.

This symbol indicates the maximum vacuum level of the pump.

This symbol indicates the flow levels of the pump.

This symbol indicates the number of items n that the content is sufficient for.

This symbol indicates the electrical specifications of the pump.

This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).

This symbol indicates alternating current.

This symbol indicates direct current.

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates the weight of the pump.

This symbol indicates a carton package.

This symbol indicates the dimensions (h x w x d) of the pump.

This symbol indicates to keep the device away from sunlight.

This Symbol indicates a Prescription Device. CAUTION: Federal US law restricts this device to sale by or on the order of a health professional.

This symbol indicates to handle the fragile device with care.

20 Technical specifications

min. vacuum, – 60 mmHg / – 8 kPa (+/– 15 %)
med. vacuum, – 100 mmHg / – 13 kPa (+/– 15 %)
max. vacuum, – 600 mmHg / – 80 kPa (– 15 %)

Measured at 0 m, atmospheric pressure: 1013.25 hPa Please note: vacuum levels may vary depending on location (meters above sea level, atmospheric pressure and temperature).

low flow: 15 l/min. (+/– 10 %)

2.0 kg / 4.4 lbs
Without jar

100–240 V, 50–60 Hz
12 V, 3.4 A, NiMH
Power adapter
UE48-120340SPA1

ISO 9001
ISO 13485
CE (93/42/EEC), IIa

h x w x d
223 x 255 x 95 mm

Transport/Storage

Transport/Storage
Rechargeable battery

Operation

IP21

21 Technical documentation

WARNINGS

Do not use other accessories than those specified or sold by the manufacturer as replacement parts for internal components as it may result in increased emissions or decreased immunity of the Clario Toni pump.

EMC

Clario Toni is EMC-tested in conformity with the requirements of IEC 60601-1-2:2007 and can be used in the vicinity of other EMC-tested devices that fulfill the requirements of the relevant IEC 60601-1-2 standard. Untested HF (high-frequency) sources, radio networks or the like can influence the operation of the device and may not be operated in combination with the system. Clario Toni is a medical device that requires special safety precautions and must be installed and placed in operation in accordance with the attached EMC information. Portable and mobile RF communication devices (mobile telephones) can affect Clario Toni.

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Electromagnetic Emissions		
Clario Toni is intended for use in the electromagnetic environment specified below. The customer or the user of Clario Toni should assure that they are used in such environment.		
Emission Tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	Clario Toni uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference with nearby electronic equipment.
RF emissions CISPR 11	Class B	Clario Toni is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Not applicable	

WARNINGS

Clario Toni should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, Clario Toni should be observed to verify normal operation in the configuration in which it will be used.

Electromagnetic Immunity

Clario Toni is intended for use in the electromagnetic environment specified below. The customer or the user of Clario Toni should assure that they are used in such environment.

Immunity Tests	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input / output lines	± 2 kV for power supply lines not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % U_T (>95 % dip in U_T) for 0.5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 sec	<5 % U_T (>95 % dip in U_T) for 0.5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of Clario Toni requires continued operation during power mains interruptions, it is recommended that Clario Toni be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels of a typical commercial or hospital environment.

Note: U_T is the a.c. mains voltage prior to application of test level.

Electromagnetic Immunity

Clario Toni is intended for use in the electromagnetic environment specified below. The customer or the user of Clario Toni should assure that Clario Toni is used in such environment.

Immunity Tests	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of Clario Toni, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m) Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. ^a Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed RF transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which Clario Toni are used exceeds the applicable RF compliance level above, Clario Toni should be observed to verify normal operation. If abnormal operation is observed, additional measures may be necessary, such as reorienting or relocating Clario Toni.
^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distance between portable and mobile RF communications equipment and Clario Toni

Clario Toni is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of Clario Toni can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and Clario Toni as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter	Separation distance according to frequency of transmitter		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
W			
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Notes

Medela AG
Lättichstrasse 4b
6341 Baar, Switzerland
www.medela.com

International Sales

Medela AG
Lättichstrasse 4b
6341 Baar
Switzerland
Phone +41 41 562 51 51
Fax +41 41 562 51 00
ism@medela.ch
www.medela.com

Australia

Medela Australia Pty Ltd,
Medical Technology
3 Arco Lane
Heatherton, Vic 3202
Australia
Phone +61 3 9552 8600
Fax +61 3 9552 8699
contact@medela.com.au
www.medela.com.au

Canada

Medela Canada Inc.
4160 Sladeview Crescent Unit # 8
Mississauga, Ontario
L5L 0A1
Canada
Phone +1 905 608 7272
Fax +1 905 608 8720
info@medela.ca
www.medela.ca

China

Medela (Beijing)
Medical Technology Co., Ltd.
Unit M, 4F
No 1068 Wuzhong Road,
Minhang District
Shanghai 201103
P.R. of China
Phone +86 21 62368102
Fax +86 21 62368103
info@medela.cn
www.medela.cn

France

Medela France Sarl
14, rue de la Butte Cordière
91154 Etampes cedex
France
Phone +33 1 69 16 10 30
Fax +33 1 69 16 10 32
info@medela.fr
www.medela.fr

Germany

Medela Medizintechnik
GmbH & Co. Handels KG
Postfach 1148
85378 Eching
Germany
Phone +49 89 31 97 59-0
Fax +49 89 31 97 59 99
info@medela.de
www.medela.de

India

Medela India private limited
c/o Vatika Business Park
First floor, tower 2,
Sohna Road, Sec-49,
Gurgaon 122 002
India
Phone +91 124 4416999
Fax +91 124 4416990
info@medela.in
www.medela.in

Italy

Medela Italia Srl
Via Turrini, 13/15 – Loc. Bargellino
40012 Calderara di Reno (BO)
Italy
Phone +39 051 72 76 88
Fax +39 051 72 76 89
info@medela.it
www.medela.it

Japan

Medela K.K.
G-PLACE Sasazuka 13F
2-26-2, Sasazuka, Shibuya-ku
Tokyo 151-0073
Japan
Phone +81 3 3373 3454
Fax +81 3 3373 3457
info@medela.jp
www.medela.jp

Netherlands & Belgium

Medela Benelux BV
Uilenwaard 31
5236 WB's-Hertogenbosch
The Netherlands
Phone +31 73 690 40 40
Fax +31 73 690 40 44
info@medela.nl
info@medela.be
www.medela.nl
www.medela.be

Poland

Medela Polska Sp. z o.o.
Ul. Lewinowska 8
03-684 Warszawa
Poland
Phone +48 22 865 12 50
Fax +48 22 864 38 65
info@medela.pl
www.medela.pl

Russia

OOO Medela
Vavilova str. 97, entrance 3
117335 Moscow
Russia
Phone +7 495 980 6194
Fax +7 495 980 6194
info@medela-russia.ru
www.medela-russia.ru

Spain & Portugal

Productos Medicinales Medela, S.L.
c/ Manuel Fernández Márquez, 49
08918 Badalona (Barcelona)
Spain
Phone +34 93 320 59 69
Fax +34 93 320 55 31
info@medela.es
info@medela.pt
www.medela.es
www.medela.pt

Sweden

Medela Medical AB
Box 7266
187 14 Täby
Sweden
Phone +46 8 588 03 200
Fax +46 8 588 03 299
info@medela.se
www.medela.se

Switzerland

Medela AG
Lättichstrasse 4b
6341 Baar
Switzerland
Phone +41 848 633 352
Fax +41 41 562 51 00
contact@medela.ch
www.medela.ch

United Kingdom

Medela UK Ltd.
Huntsman Drive
Northbank Industrial Park
Irlam, Manchester M44 5EG
UK
Phone +44 870 950 5994
Fax +44 870 389 2233
info@medela.co.uk
www.medela.co.uk

USA

Medela Inc.
1101 Corporate Drive
McHenry, IL 60050
USA
Phone +1 877 735 1626
Fax +1 815 307 8942
suction@medela.com
www.medelasuction.com