Clario

HOME CARE PUMP

- EN** INSTRUCTIONS FOR USE
- DE** GEBRAUCHSANWEISUNG
- FR** MODE D'EMPLOI
- IT** ISTRUZIONI PER L'USO
- NL** GEBRUIKSAANWIJZING
- SV** BRUKSANVISNING
- FI** KÄYTTÖOHJEET
- NO** BRUKSANVISNING
- ES** INSTRUCCIONES DE USO
- PT** MANUAL DE INSTRUÇÕES

074.0001

074.0002

074.0003

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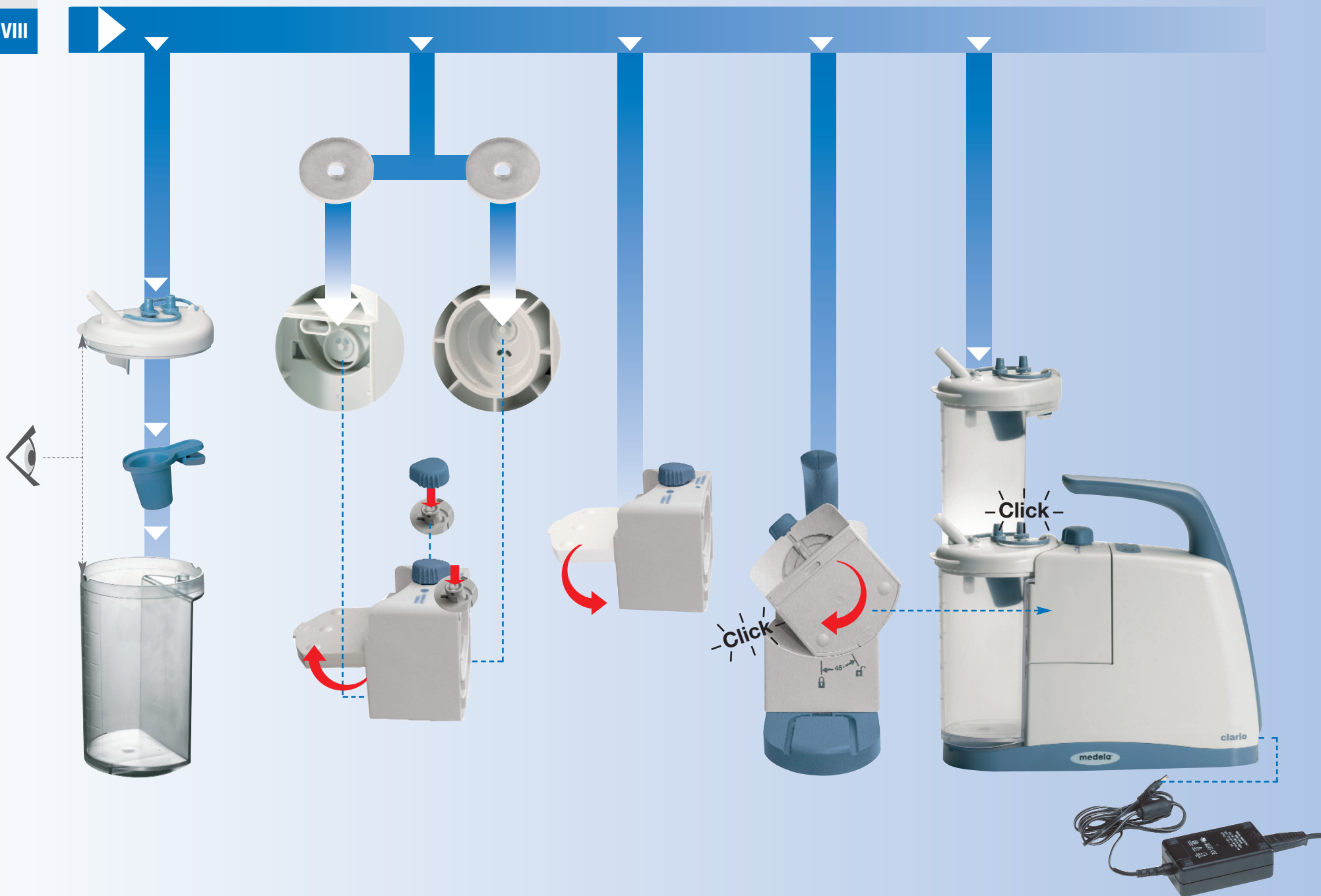
I Congratulations

The Medela Clario is a high quality suction pump for use on order of a health professional/physician only. This product provides you with a unique threefold hygienic design and optimal protection from an overflow. It is operated by a membrane system with an electric and maintenance-free motor. Using the 3 vacuum settings, the suction strength can be adjusted to the required level.

II Use in accordance with the instructions

Use only according to the instructions from your doctor and after getting instructions from trained medical specialists.

Indications The Clario is intended for use in the medical field for suctioning secretions, body liquids and foreign objects in the nasal, pharyngeal and tracheal areas. This suction pump is suitable for use at home, in the hospital, in the car (AC/DC version) or outdoors (AC/DC version).

Contra-indications The device is not suitable for use for prolonged suctioning.

III Important safeguards

Correct and safe operation only with Medela accessories

Read all safety instructions before use.

For patients for whom a breakdown of the device can lead to a critical situation (e.g. patients with acute dyspnoea or severe catarrhal congestion), a replacement device must always be available.

- Before you plug in the device, please check that your local power supply is the same as the voltage given on the specification plate.
- In each of the following cases, the device must not be used and it must be repaired by Customer Services:
 - If the power cable or the plug are damaged.
 - If the device is not functioning perfectly.
 - If the device is damaged.
 - If the device indicates clear safety defects.
- The pump must stand upright during use.
- The device must not be used for suctioning explosive, easily flammable or corrosive liquids.
- The connecting tubing supplied with the device must never come into direct contact with the suction area.
- Keep the power cord away from hot surfaces.
- The plug and the ON/OFF switch must not come into contact with moisture.

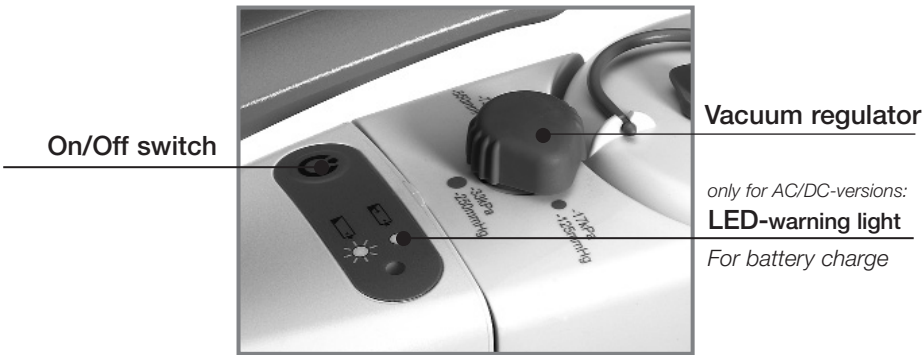
- Separation from the mains is only assured through the disconnection of the plug and socket connection.
- Never pull the plug out of the mains socket by pulling on the connecting cable!
- Never use the device at high room temperatures, while bathing or showering, if you are very tired or in an environment where there is a risk of explosion.
- Never leave the device unattended when it is switched on.
- The Medela Clario is a medical device that requires special safety measures in regard to EMC and must be installed and put into operation in accordance with the attached EMC information.
- Never place the device in water or other liquids.

These Instructions for Use must be kept for later reference.

IV Guarantee

2 years for the unit from date of delivery, battery: 6 months. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage as well as use by unauthorized persons.

V Operating elements

Other elements (see inside of cover):

- | | |
|--------------------|---|
| a Motor unit | h Floater |
| b Membrane | i Pug for closing lid |
| c Handle | j Suction port |
| d SafetyChamber | k Disposable connection tubing with fingertip control |
| e Vacuum regulator | l Rechargeable battery |
| f Valve discs | m Power cord |
| g Canister | |

VI Spare parts/Accessories

Correct and safe operation only with Medela accessories

Contained in the delivery package

- j 074.0003 Disposable connection tubing with fingertip control

Optional accessories – not contained in the delivery package

- 074.0007 Carrying bag
- 074.0017 Car connection cable for 12 V DC

VII AC/DC versions

Indicators	 visual	LED display
during battery operation	does not light up	Rechargeable battery is fully charged / empty
	blinks slowly	Rechargeable battery soon empty (~10 mins. pumping duration)
During mains operation	lights up	Rechargeable battery is being charged
	does not light up	Rechargeable battery is fully charged
	blinks rapidly	Rechargeable battery is defective. Replace with a new one.

Performance of rechargeable battery

Duration of pumping

at max. vacuum

at min./med. vacuum

> 50 mins.

< 50 mins.

Charging time (charged up to 100%)

~ 5 hrs.

Rechargeable battery

1. Disconnect the mains cable
2. Select maximum vacuum
3. Switch on the Clario
4. Measure the running time
5. Compare the running time

> 50 mins.

OK, if the above value is reached

not OK

- Calibrating the rechargeable battery
- Repeat this process

If the calibration was not successful, **replace battery**

NiMH rechargeable battery

Before first use

after storage
duration > 2 months

Calibrating the rechargeable battery

A Completely discharge the rechargeable battery

- Disconnect the mains cable
- Switch on the Clario and allow to run until it comes to a standstill

B Charge the Clario

- Plug in the mains cable – the rechargeable battery is being charged
- Wait until the charging indicator goes out

Repeat this process once
The Clario is then ready for use

Operation with rechargeable battery

- Only charge the rechargeable battery when the charging indicator blinks slowly

Storage

In order to keep the self-discharge at a minimum, store the Clario and the replacement rechargeable batteries at temperatures below 25°C (77°F). When storing the Clario over a longer period of time, we recommend to store device and battery separately.

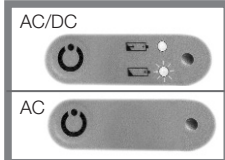
VIII Application after assembly (see front flap)

Wear gloves for all operations.

Use only after getting instructions from trained medical specialists.

1. Check general condition of the device

- Membrane (has no tears and is clean)?
- Floater attached and OK?
- Canister and lid - cracks, brittle areas, discolorations? If necessary, replace
- Valve discs in SafetyChamber? Tight seal?
- Check markings at canister and lid.


2. Test function


- AC/DC versions: before first use, calibrate the battery (see chapter VII)
- AC versions: plug in

a. Switch the device on

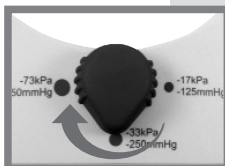
b. Set maximum vacuum

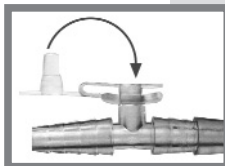c. Attach fingertip tubing and plug fingertip port

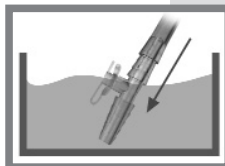
d. Place fingertip into water and check suction or use the optional vacuum gauge

ok?

▶ **not OK**, see Chapter X

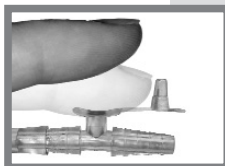OK

3. Use

- Attach catheter prescribed by physician
- Adjust the vacuum for use according to instructions from a specialist
- Do not let the skin cling! Release vacuum (see picture)
- Automatic stop at max. fluid level of 550 ml
- Empty canister regularly

IX Cleaning

After each use:

- Canister**
- Remove
 - Secure connectors before transport
 - Empty
 - Disassemble parts
 - Clean in dishwasher, boiling water or disinfecting solution.

- Tubings/ Catheter**
- Remove and dispose of according to local regulations.

Clean SafetyChamber after an overflow or when the device has not been used over an extended amount of time.

- Safety Chamber**
- Remove and disassemble
 - Clean in dishwasher, boiling water or disinfecting solution.

If necessary:

- Housing/ Membrane**
- Unplug and switch off the device before cleaning
 - Clean the device with a damp cloth and with non-aggressive cleaning material.

X Troubleshooting

The device does not contain any parts which may be repaired by the user!

Motor not running

Check

- Is there mains/battery power present?
- Is the device switched on?

Insufficient vacuum

Check

- Check general condition of the device according to VIII, 1
- The setting of the vacuum regulator
- The condition of the tubings
- Tight seal of valve disks inside safetyChamber
- Membrane is clean and has no tears
- If canister, lid and seals have tears, brittle areas or blockages

Running time from battery insufficient

Calibrating the rechargeable battery => Chapter VII

Too loud Check position and seal of valve disks inside SafetyChamber

If the fault cannot be corrected, contact your service center

XI Replacing the rechargeable battery

Disconnect the pump from the mains supply

- 1) open lid
- 2) Replace the rechargeable battery (use only Medela battery)
- 3) Plug in battery
- 4) Close lid

Corresponding illustrations are to be found on the back flap

XII Routine check

The Clario Home Care Pump requires no maintenance, but please perform the functional tests before each use. There are no parts in the Clario motor-unit that can be repaired by the user.

XIII Disposal

The Clario is made from various metals and plastics. Before disposal, the device is to be rendered unusable and it must be disposed of in accordance with the local regulations. With the AC/DC versions, the electronic components and the rechargeable batteries must be disposed of separately, in accordance with the local regulations.

XIII Parts list

	Article numbers
a1 Motor unit AC-version	600.0798
a2 Motor unit AC/DC-version	600.0799
d SafetyChamber	074.0002
e Vacuum regulator	810.0627
f Valve discs	074.0008
g Canister set	074.0001
h Floater	074.0010
i Plug for closing lid	810.0639
k Disposable connection tubing with fingertip control	074.0003
l Rechargeable battery	074.0005
m1 Power cord (EURO)	928.0050
m2 Power adapter (EURO)	920.0037
m3 Power cord (USA)	928.0048
m4 Power adapter(USA)	920.0035
m5 Power cord (GB)	928.0052
m6 Power adapter (GB + AUS/NZ)	920.0037
m7 Power cord (AUS/NZ)	928.0051

*The company reserves the right to change technical specifications without prior notice.
Product may vary from picture.*

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Medela UK Ltd.
Medela, Inc.
Medela AG, Medical Technology